


SNOEZELEN® INTERACTIVE AROMA PANEL

Item# 2120417

INSTRUCTION MANUAL

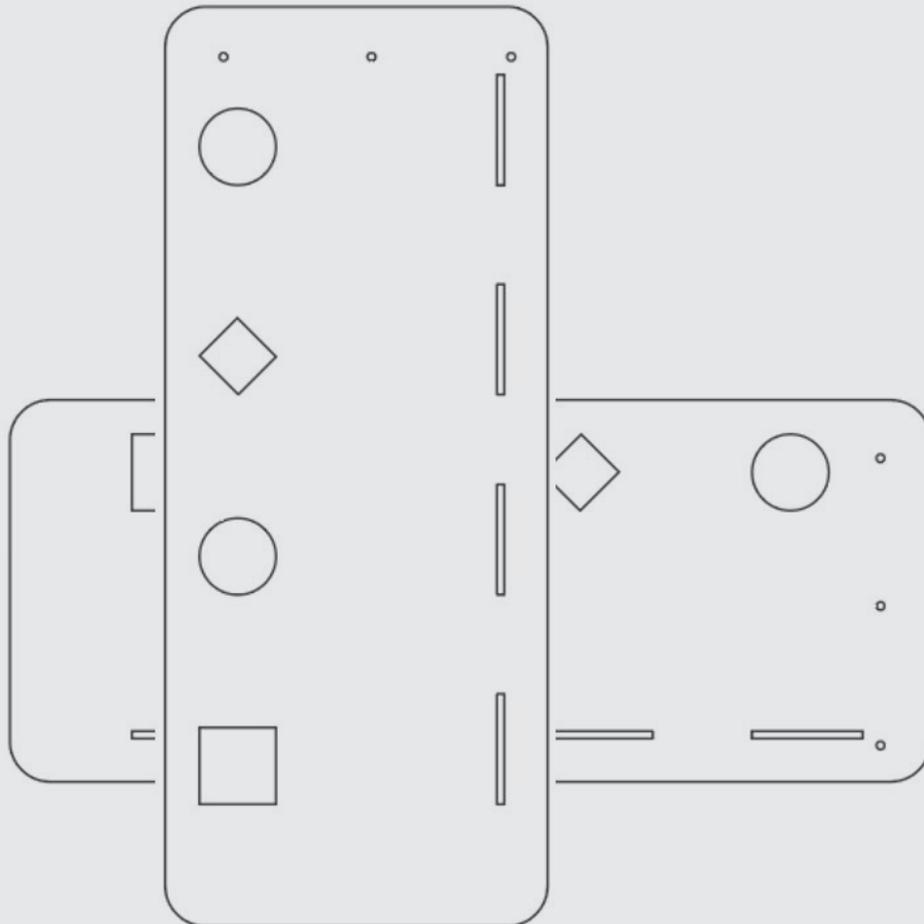

PRODUCT DESCRIPTION	3
USER BENEFITS	3
PRODUCT COMPATIBILITY	3
INCLUDED PARTS.....	3
TECHNICAL SPECIFICATION	4
PRODUCT SAFETY & CARE INSTRUCTIONS	5
PRODUCT ASSEMBLY GUIDE/INSTALLATION.....	5
TROUBLE SHOOTING.....	5
USER GUIDE	6

PRODUCT DESCRIPTION

Activate scents, colors, sounds, and breezes for aromatic, visual, auditory, and tactile stimulation. LED lights indicate which scent is being diffused. Scents to be purchased separately, School Specialty does not offer aromas. The integral fans feature two speed settings. Designed for wall mounting (horizontally or vertically). Available in frosted opal finish only.

USER BENEFITS

- The aroma panel engages the visual sense with colorful touch-sensitive buttons
- Audible stimulation through the button sounds when pressed
- Engages an understanding of cause and effect through button presses that release the aromas
- Helping the recognition of actions leading to specific outcomes, aiding in problem-solving and learning.

PRODUCT COMPATIBILITY

The Interactive Aroma Panel is fully compatible with the following products:

2157413 Sensory Magic™ 3

2119877 Sensory Magic™ 2

2120147 Snoezelen® Wi-Fi Wireless Controller

2120277 Mini Talking Cube

2120037 Color Command Centre™

INCLUDED PARTS


- 1x Interactive aroma panel
- 1x Scent pads
- 1x External Power supply, UL/CSA Listed with UK, EU and North American Power Cords
- 4x Aroma boxes (that house the aromas)

TECHNICAL SPECIFICATION

Weight: 26.5lbs 12kg

Dimensions: 23½" x 10" x 4" (60 x 25 x 10.5cm)

Power: 12v 60w, 5A)

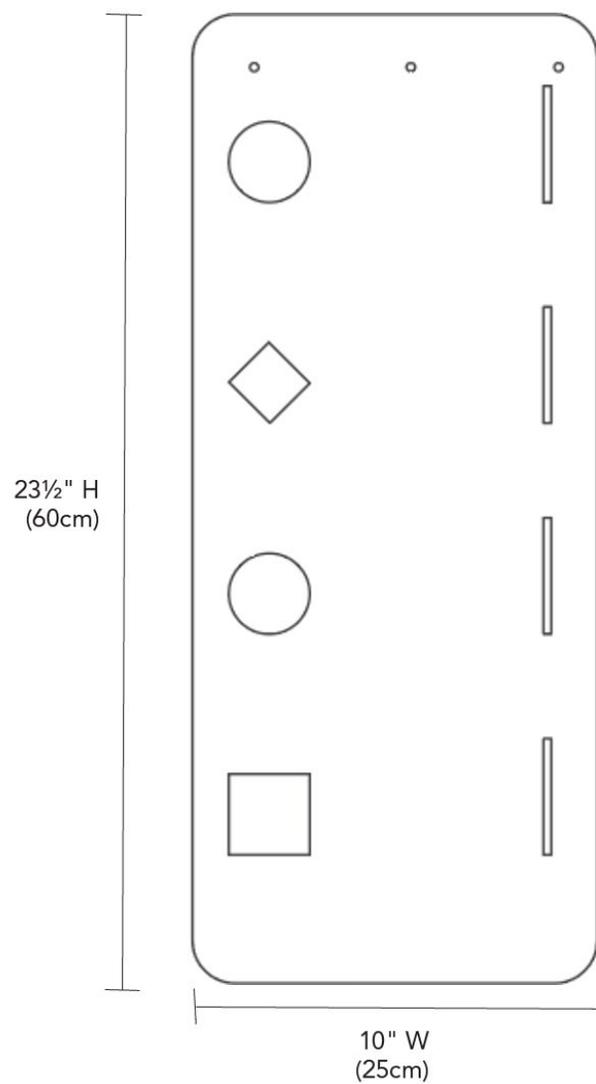

PRODUCT SAFETY & CARE INSTRUCTIONS

Safety:

- Supervise the use of this product.

Care and Maintenance

- Wipe clean with a soft, very slightly dampened cloth.
- Do not use abrasive cleaning products or bleach.

PRODUCT ASSEMBLY GUIDE/INSTALLATION

Installation

Wall-mounting- Use the pre-drilled holes (one at each of the corners) to wall-mount the panel. No fixings are included in this product for wall-mounting. When fixing it to the wall please use the appropriate hardware for your wall construction type. We recommend using toggle bolts for drywall or Tapcons for cinderblock concrete construction. Not all construction types are the same. If you are unsure, please consult a qualified professional.

TROUBLE SHOOTING

- No power

Please ensure that the receptacle has power. Check connections are secure in both the receptacle and on the panel.

USER GUIDE

Operation

1. Plug the panel power supply into the panel before plugging it into the receptacle.
2. Turn the panel on using the on/off switch on the left-hand side of the panel. The four-shaped buttons will then light up in four different colors.
3. Press one of the four shaped buttons and be rewarded by sound, color, breeze and aroma. A “spraying” sound is played as the aroma is blown out by the internal fan. When pressed, the button color changes to green. The button stays green, and the internal fan remains on for the duration of the button press: Press and hold the button for as long as desired.

The button will revert to its original color if not pressed.

Filling cartridges with aroma

1. Unlock the front of the Interactive Aroma Panel and unscrew the two black securing knobs to slide open the lid.
2. Remove the four aroma cartridges held by magnets.
3. Fill cartridges with scented beads, sachets, or aroma oils. Use essential oils sparingly on pads/sponges, avoiding direct contact with plastic.
4. Refit the cartridge lid or vent and place the cartridges back, aligning the numbered magnets.
5. Slide the lid back and retighten the knobs.

Wi-Fi Operation

The Interactive Aroma Panel may be used with other Snoezelen® compatible Wi-Fi products. Simply press any Snoezelen® Wi-Fi switch, and the Interactive Aroma Panel buttons will change to the color selected. The Wi-Fi color will remain for one minute, unless another Wi-Fi color is chosen. The Interactive Aroma Panel buttons then revert to their original colors.

Sensory Magic

The Interactive Aroma Panel is compatible with Sensory Magic™ 2 and 3 and can be controlled from within the Sensory Magic™ system.

