



Classroom Select® Recovery Lounge

ASSEMBLY INSTRUCTIONS


Thank you for purchasing Classroom Select® Recovery Lounge from School Specialty®. Please review the enclosed packing list to ensure all components are included and read the instructions carefully before you begin assembly. If you find anything missing or have any questions now or after set-up, please contact Customer Care at 888.388.3224 or customer care.schoolspecialty.com/contact-us-email.

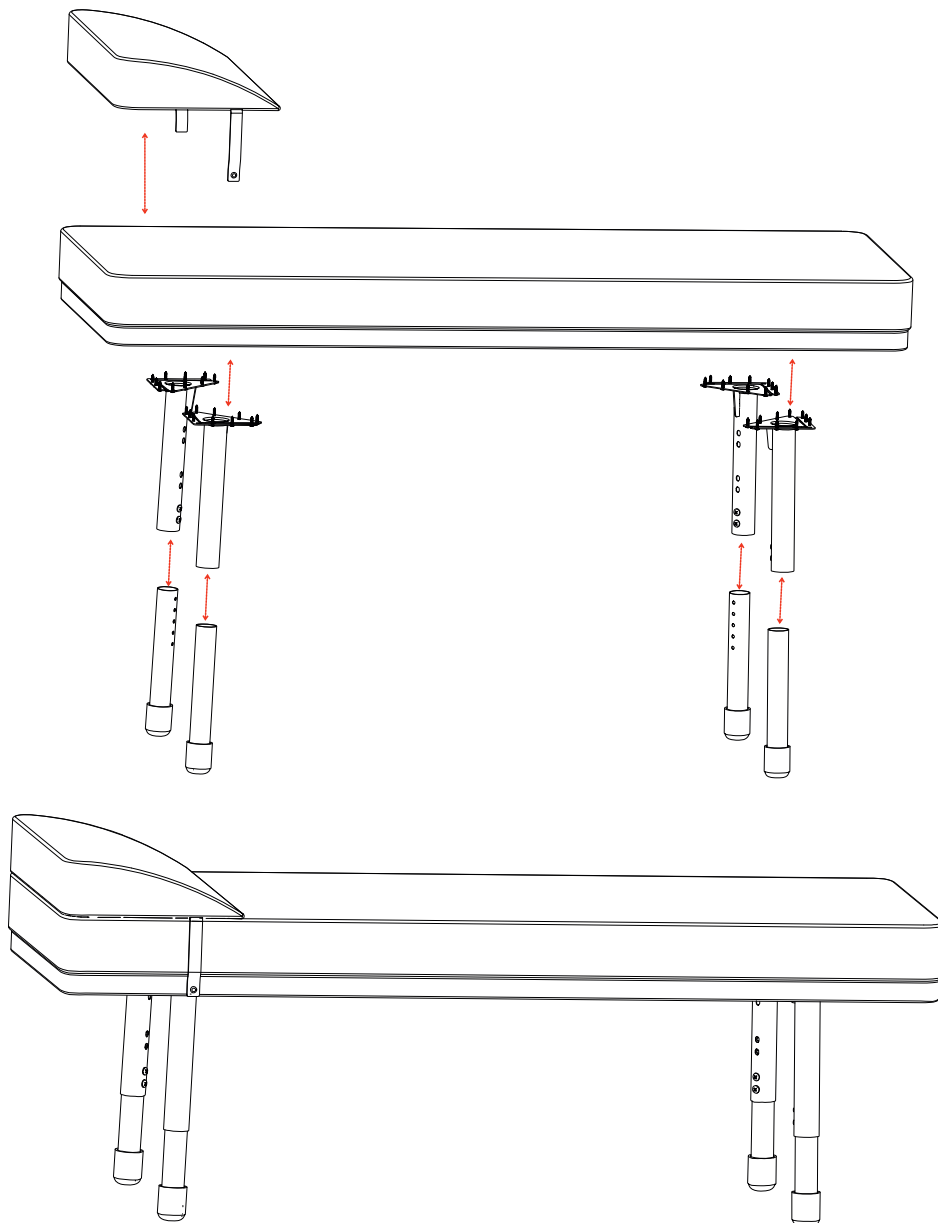


PART AND TOOLS LIST

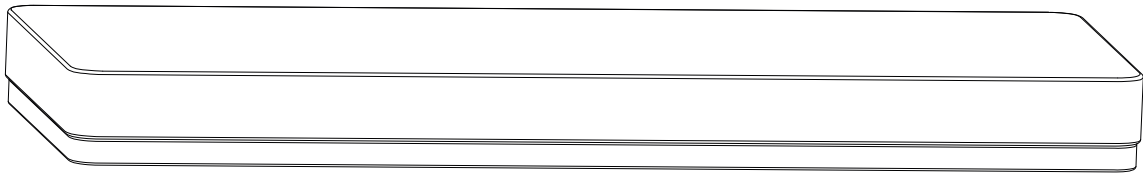


Royal Recovery Lounge
Quantity: 1

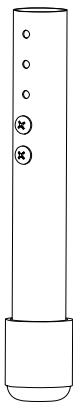


Apollo® Leg Insert
Quantity: 4

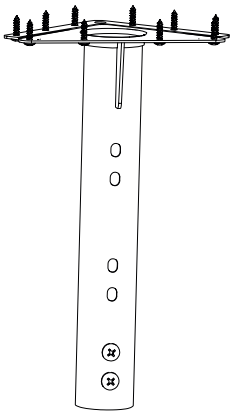

Apollo® Leg Upper
Quantity: 4

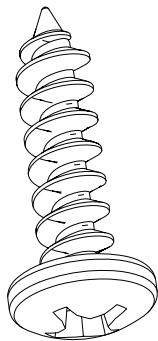


Phillips Rounded Head Screw
Quantity: 16

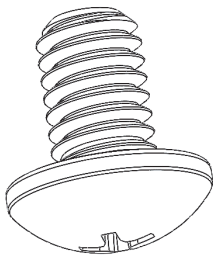

Serrated Bolt
Quantity: 8


STEP 1

To begin the installation, place the rectangular base on a stable, non-abrasive soft surface to avoid any scratching, with the mounting brackets facing upward, ensuring proper alignment. Next, position each vertical support pole over the designated mounting areas, aligning the pre-drilled holes on the base with those on the bottom flange of the poles. As shown in the zoomed in image (1) slide the leg upper into the v-bracket. Once aligned, use the provided screws to securely fasten the support poles to the base (shown in zoomed image 2), tightening each screw evenly to ensure stability on all four legs. After securing the screws, perform a final inspection to confirm that the support poles are firmly attached and properly aligned. Ensuring all components are securely fastened will provide a stable foundation for the next steps in the assembly process.

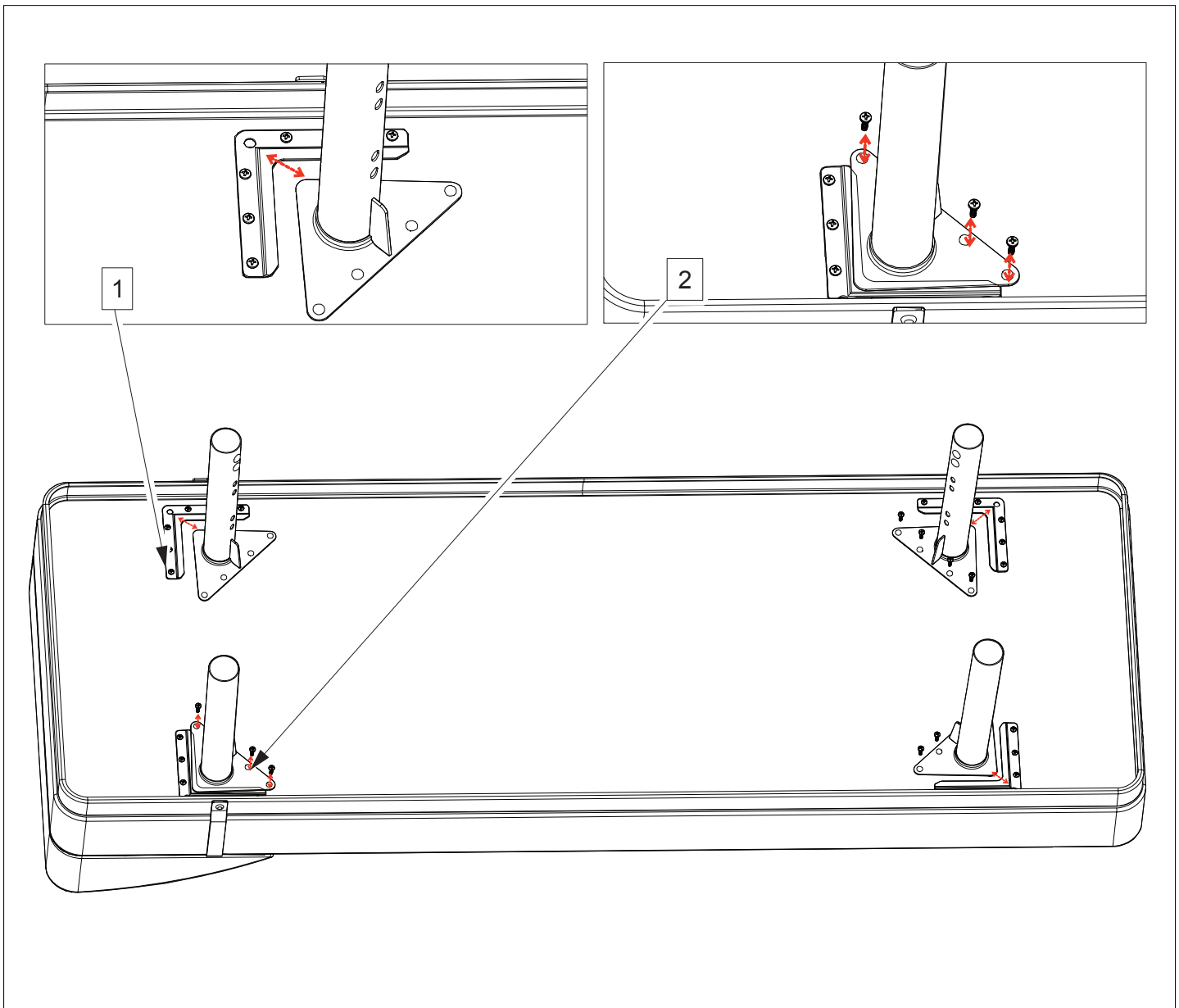


STEP 2

In this step of the installation, the leg inserts are inserted into the pre-installed leg uppers on the base. Begin by aligning the leg inserts (labeled as 1) with the open end of the corresponding leg upper, ensuring that the adjustment holes are properly aligned for height customization. Carefully slide the insert down into the leg upper, making sure it fits securely. Next, as shown in the detailed view (labeled as 2), insert and fasten the locking bolts to secure the leg insert in place. After setting the leg height, leave the leg insert screw $\frac{1}{4}$ turn loose, flip the recovery lounge over, and make sure the unit is level. Then turn $\frac{1}{4}$ turn and tighten the screws, so the inserts are not wobbly.

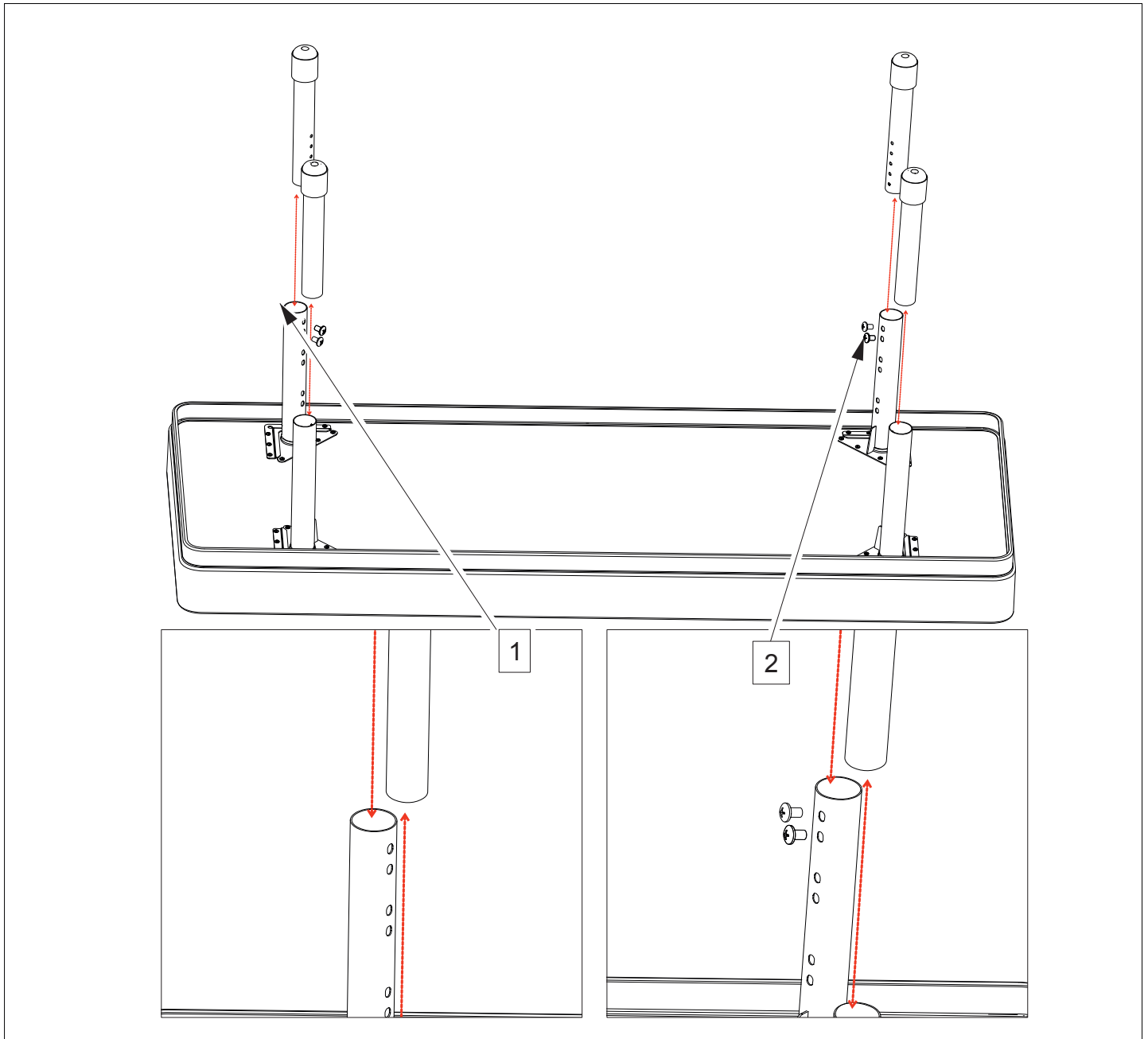


STEP 3


In step three of the installation process, the pillow is attached to the main frame of the recovery lounge. The illustration shows the pillow being positioned above the lounge, aligning its support legs with the designated mounting holes on the frame. The red arrow indicates the downward movement of the pillow, guiding it into place securely. The close-up view provides a detailed perspective of the attachment point, ensuring proper alignment. Once the pillow is inserted, it should be fastened firmly to maintain stability and comfort.

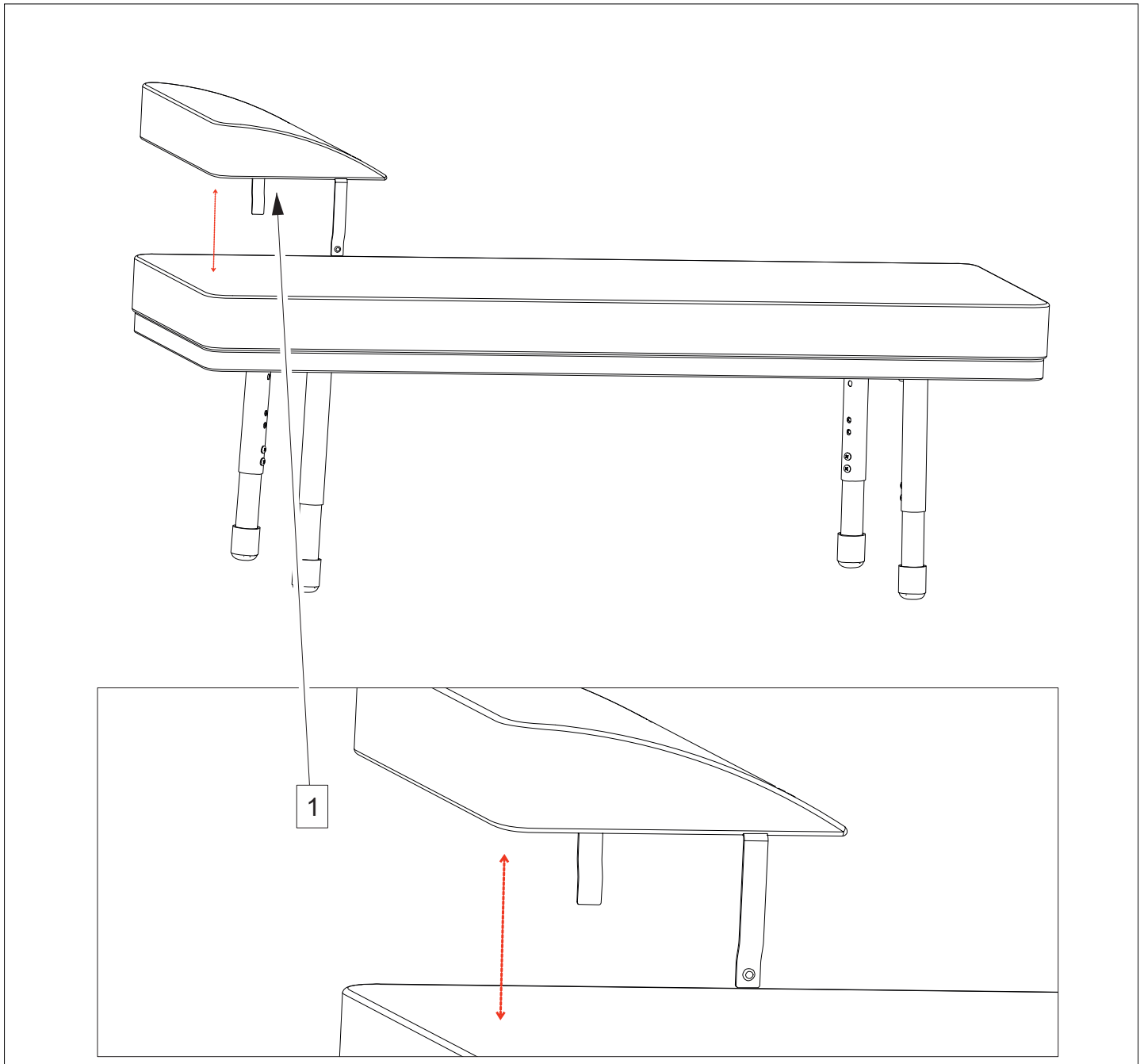


Classroom Select® Paper Roll Dispenser for Recovery Lounge (sold separately)

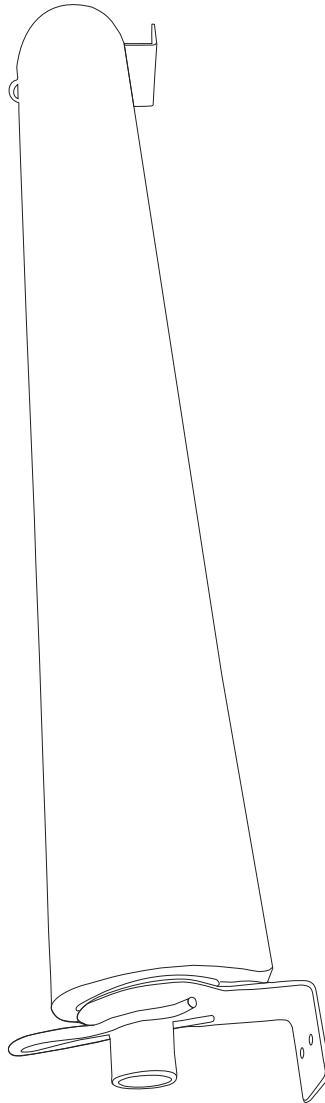


PART AND TOOL LIST

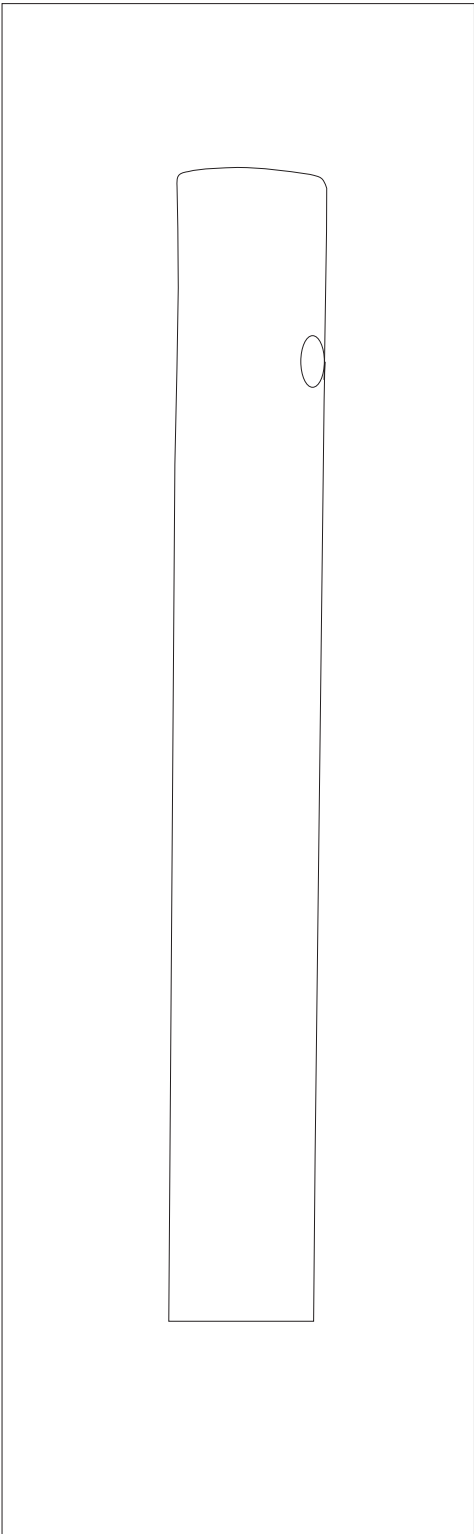


Dispenser Tube
Quantity: 1

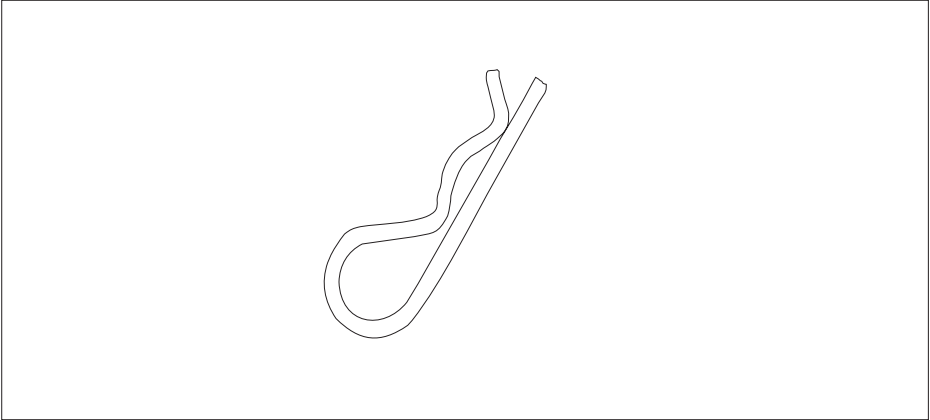

Pins
Quantity:2

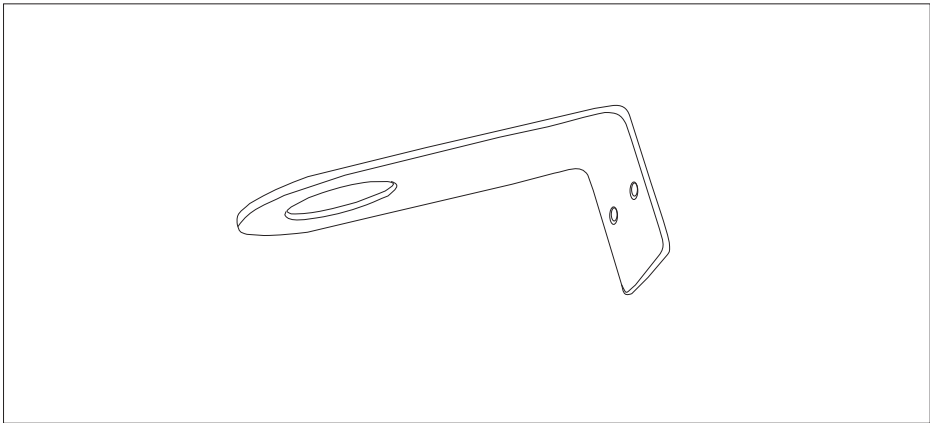


Brackets
Quantity:2

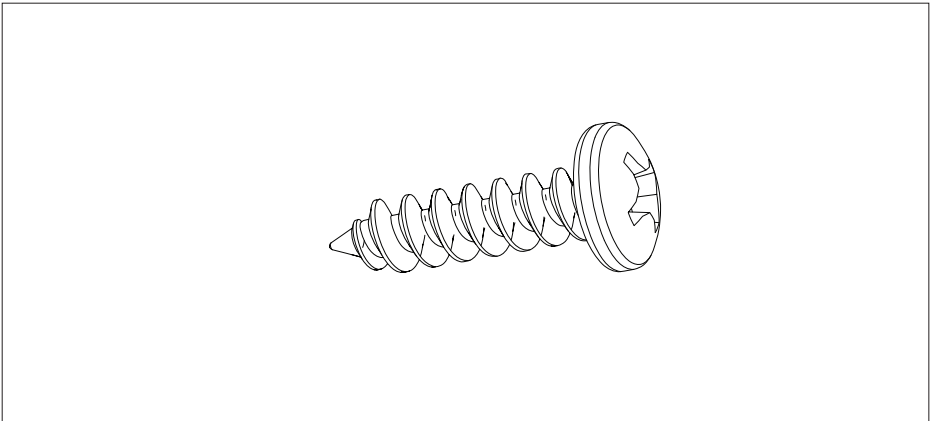

Screws
Quantity:4


STEP 1

Install one pin and one bracket on the dispenser tube.

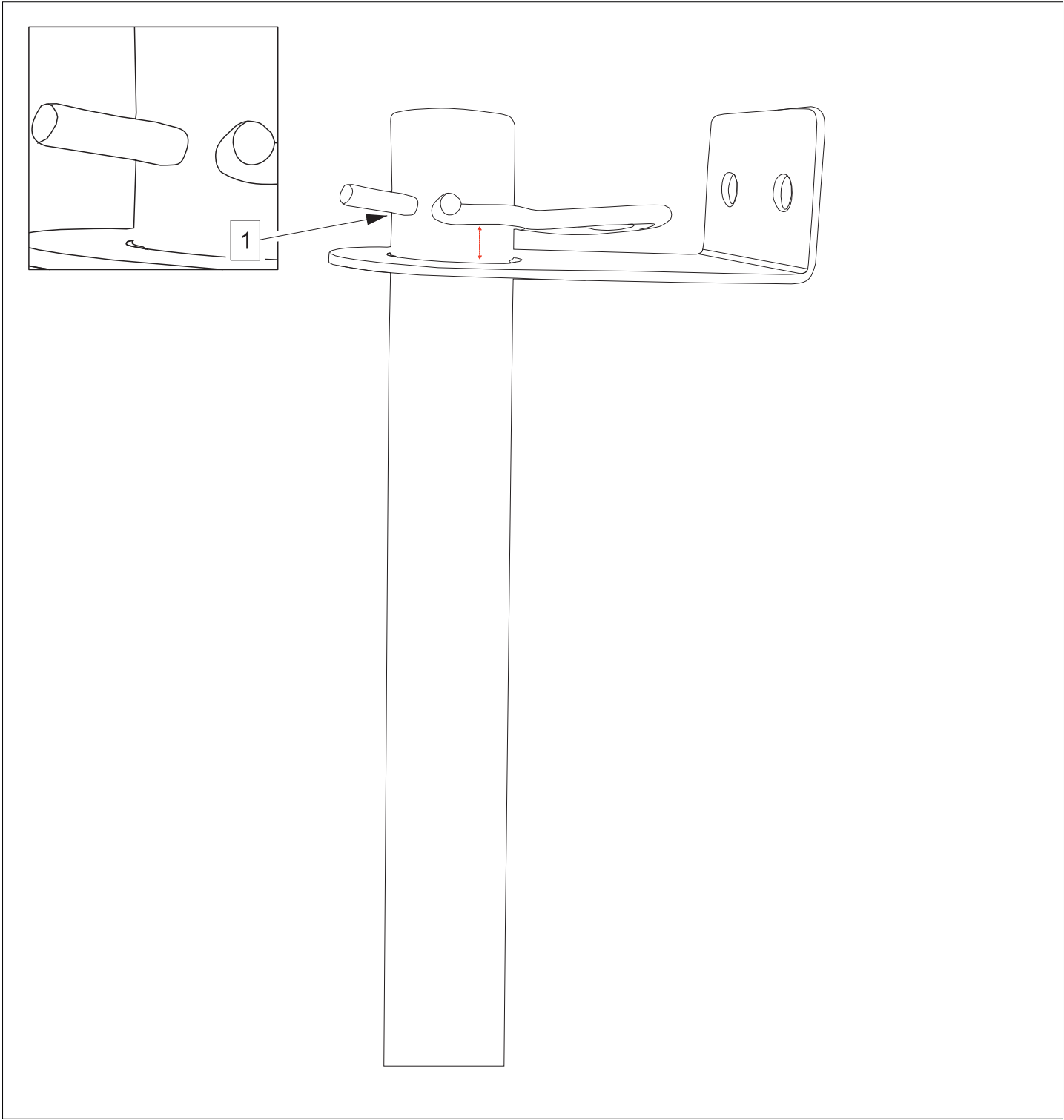


STEP 2

Slide the paper roll over the dispenser tube.

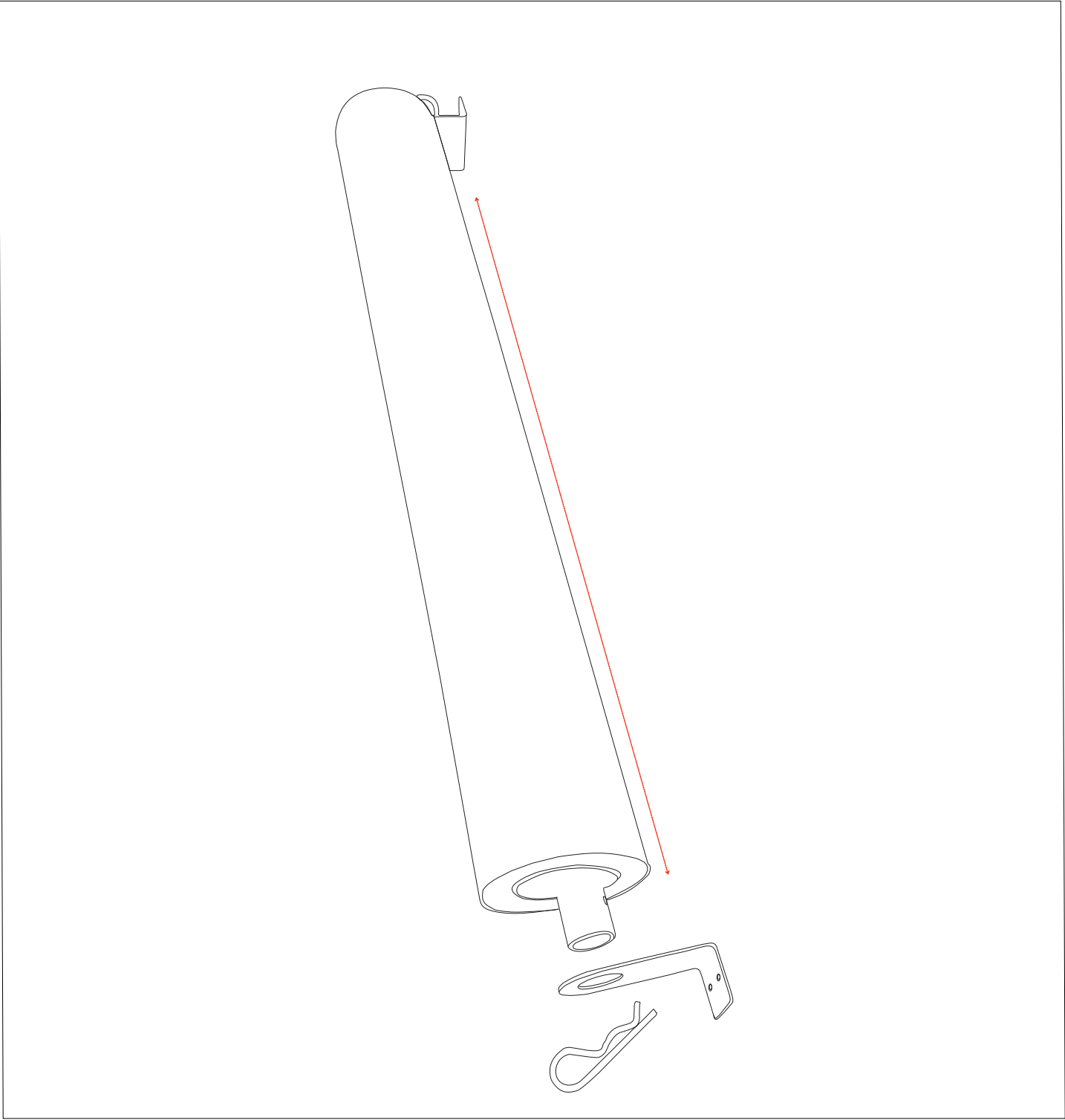


STEP 3

Install the second pin and bracket.

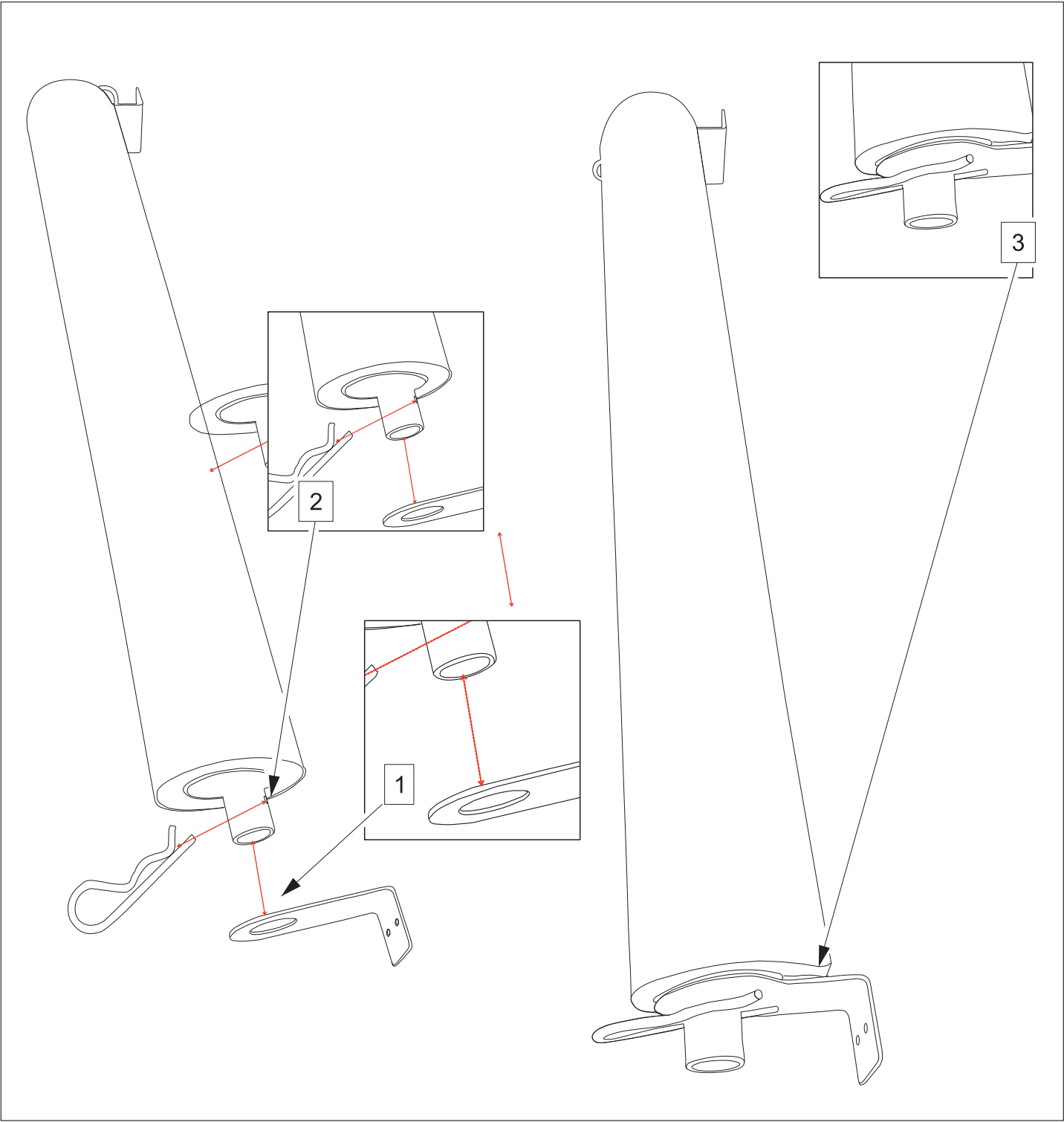

STEP 4

Place the assembled unit on the bottom of the Recovery lounge and secure in place with the four screws.

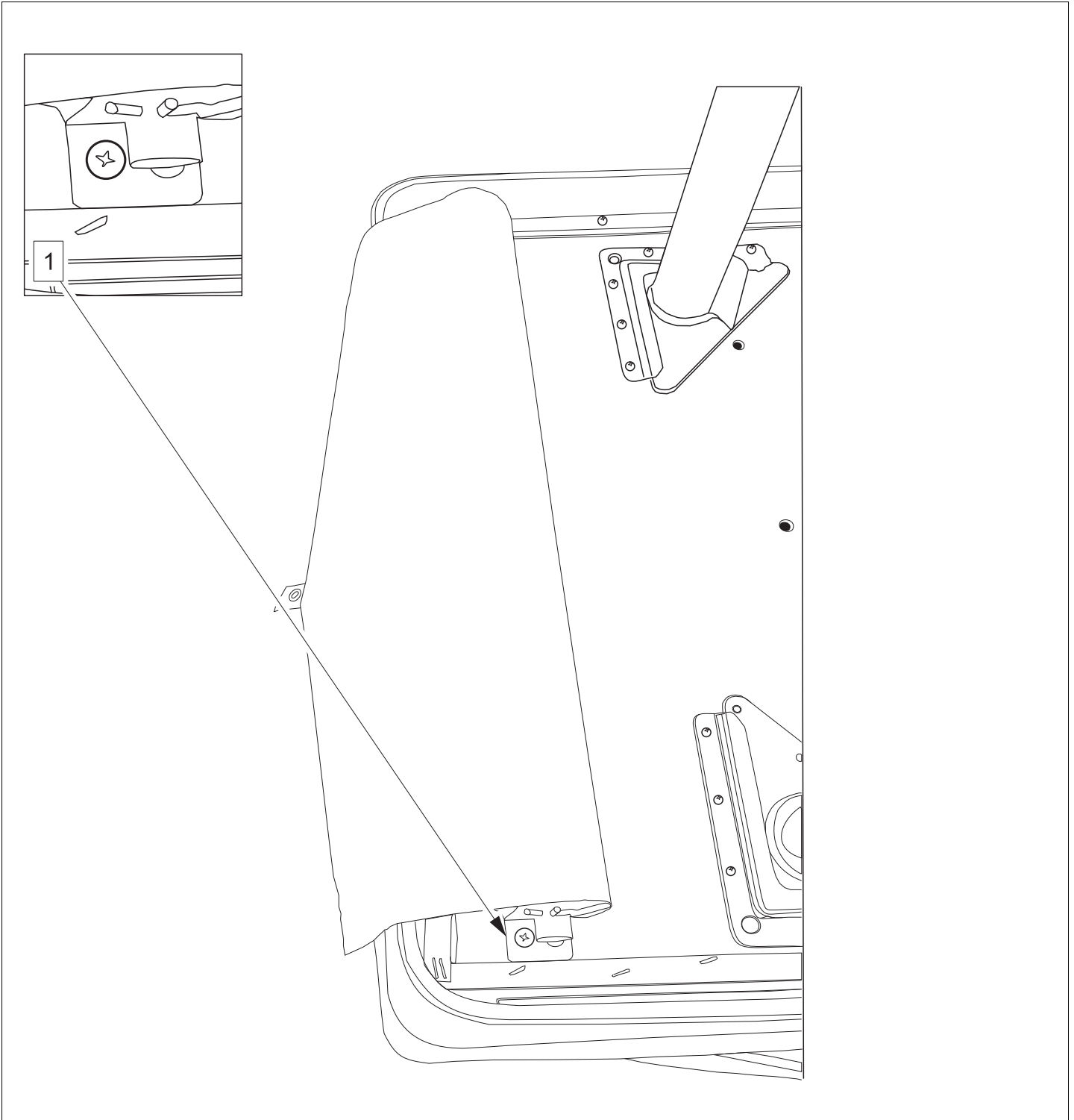

DISCLAIMERS

Please review the enclosed packing list to ensure all components are included. Please read the instructions carefully before you begin assembly. If you find anything missing or have any questions now or after set-up, please contact Customer Care at 888.388.3224 or customercare.schoolspecialty.com/contact-us-email.

Heavy or large furniture may require two or more people for proper assembly. Improper assembly may void the warranty. Ensure all hardware is correctly installed and screws are fully tightened before use. Recheck and tighten screws at least once per year. Failure to assemble or secure parts properly may result in product failure or personal injury.

The products illustrated/photographed in this assembly instruction may vary slightly from the actual product.

Product Warranty: help.schoolspecialty.com/s/article/Product-Warranties